

# Medis QFR<sup>®</sup>

## Quantitative Flow Ratio




3D model reconstruction for anatomical and physiological data

QFR<sup>®</sup> is Medis' proprietary solution delivering image-based functional assessment of coronary obstructions from standard coronary angiograms. The novel QFR approach leverages advanced computational fluid dynamic principles to accurately<sup>1</sup> and safely<sup>2</sup> assess physiologic significance of a coronary arterial segment.

### Streamlined Workflow

- Pullback curve and display each lesion assessment to help prioritize for intervention in serial lesions
- Flexible workflow with online and offline assessment capabilities

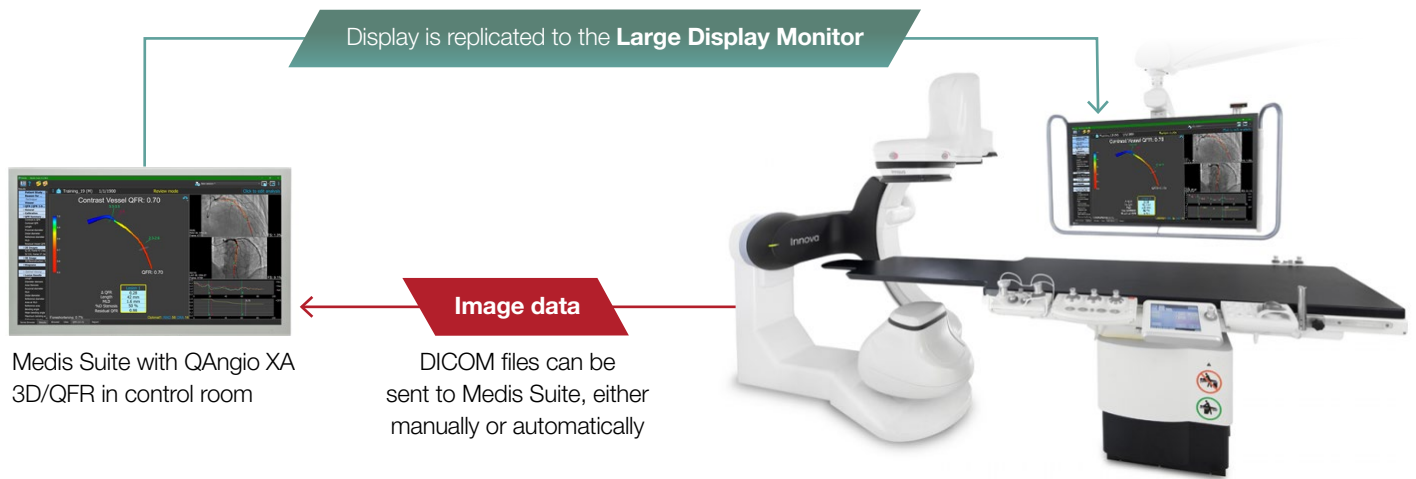

### Clinically Validated<sup>3</sup>



**>12,000**  
Patients





**>16,000**  
Lesions



**80+**  
Peer Reviewed  
Publications

# Excellent correlation and diagnostic accuracy when compared with FFR

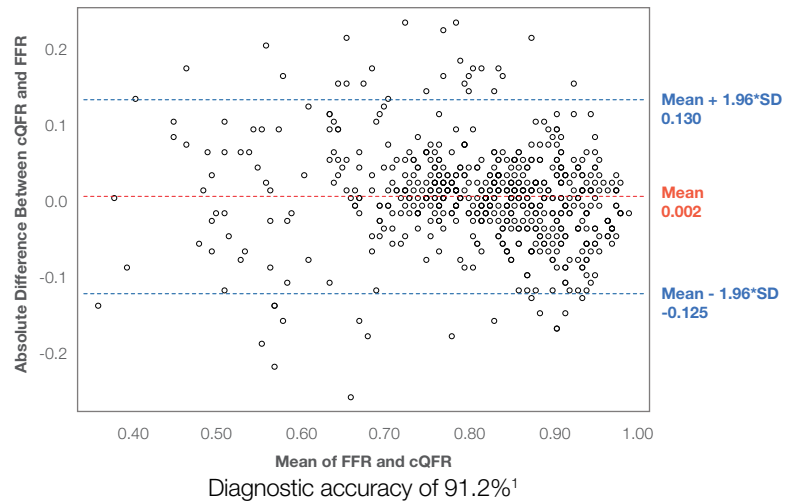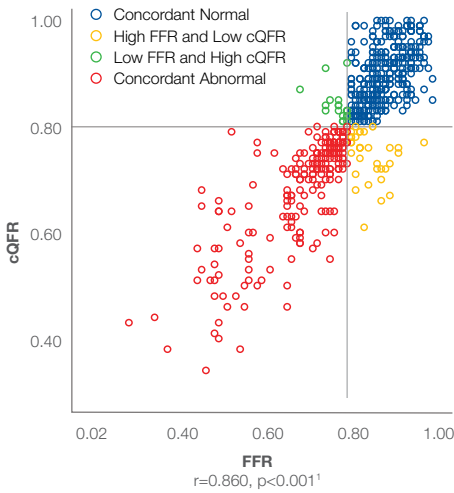

## Strong diagnostic performance compared to iFR<sup>4</sup>

	QFR ≤ 0.80	iFR ≤ 0.89
<b>Accuracy</b>	94%	74%
<b>Sensitivity</b>	97%	74%
<b>Specificity</b>	87%	74%
<b>Positive Predictive Value</b>	94%	86%
<b>Negative Predictive Value</b>	93%	56%

FFR < 0.80 reference standard

## Rapid Procedure Time<sup>5</sup>



## Improve Patient Procedural Experience

- Non-invasive = No adenosine, No pressure wire
- Reduced risk profile\*
- Reduced total procedural cost\*\*

1. Choi et al; Clinical relevance and prognostic implications of contrast quantitative flow ratio in patients with coronary artery disease. *International Journal of Cardiology*. Sept 2020  
 2. Arturo Cesaro, Felice Gragnano et al; Fractional assessment of coronary stenosis; an overview of available techniques. 2018 July 11, ruitao Zhang et al; Meta analysis of Diagnostic Performance of contrast Fractional Flow Reserve versus Quantitative Flow Ratio for Functional Assessment of Coronary Stenoses *Journal of Interventional Cardiology*; 1155/2020/732150  
 3. Data on file - PN: 35-100018  
 4. Emori H, Kubo T et al; Quantitative flow ratio and instantaneous wave-free ratio for the assessment of the functional severity of intermediate coronary artery stenosis. *Coronary Artery Disease* 2018  
 5. Westra J, Andersen BK et al; Diagnostic performance of in-procedure angiography-derived Quantitative Flow Reserve compared to pressure-derived Fractional Flow Reserve: The FAVOR II Europe-Japan Study *J Am Heart Assoc* 2018

\* When compared to FFR with adenosine induced hyperemia. No required invasive equipment that may cause cardiac injury.  
 \*\* Cost savings based on the reduced cost of QFR compared to conventional FFR and the respective cost of administration of hyperemic agent (adenosine) and additional invasive procedural resources.

Medis and QFR® are registered trademarks of Medis Associated BV. ACIST is a registered trademark of ACIST Medical Systems, Inc. QFR is delivered by Medis in strategic collaboration with ACIST Medical Systems, Inc. ACIST Medical Systems, Inc. reserves the right to modify the specifications and features described herein or discontinue manufacture of the products described at any time without prior notice or obligation. Please contact your authorized ACIST representative for the most current information.